

Reset Patient ID on BBraun Machine

To reset a patient's ID on BBraun machine:

1. Uncheck the **Start Time** from the **Pre-Treatment** tab in **Real-Time Charting**
2. Click "**Red Man Off**" on the **BBraun** machine
3. Enter new patient ID
4. Click on the "**Green Man**" to restart treatment
5. Return to **Real-Time Charting**, recheck **Start Time**, and adjust to reflect actual start time

Important Note: Old Vital Signs will not pull, but the vital signs from the point the new patient ID was entered will pull.