




PD Disinfecting Wipes Product Grid

PD Disinfecting wipes	Use	Contact Time	Image of product
<p>Purple Top Super Sani-Cloth® Germicidal Disposable Wipe</p>	<p>Surfaces and equipment</p>	<p>2 minutes—surface needs to be in contact with disinfectant</p>	
<p>Orange Tops Sani-Cloth® Bleach Germicidal Disposable Wipe</p>	<p>Bleach wipe for C Diff and blood spills.</p>	<p>4 minutes —surface needs to be in contact with disinfectant</p>	
<p>Blue Top Sani-Hands® Instant Hand Sanitizing Wipes</p>	<p>Sani-Hands for cleaning patient skin for cannulation</p>	<p>use until surface is cleaned and allow to air dry —similar to hand sanitizer</p>	
<p>White Top Easy Screen® Cleaning Wipe</p>	<p>Electronics and screens Removes dirt, grime, fingerprints, and smudges</p>	<p>use until surface is cleaned and allow to air dry</p>	